

T-5898

# AccuSign® DrinkChek

One-Step BZO Test

- *Fast: Results in 3 minutes*
- *Simple: Just One Step*
- *Convenient: Test Anywhere*

**PBM**

For Liquid Testing

Not to be taken internally.

---


Benzodiazepines are a class of widely prescribed central nervous system(CNS) depressants. They have medically useful properties, such as anti-anxiety, sedative, anti-convulsant and hypnotic-effects. However, certain types of Benzodiazepines could be abused for date rape or robbery by putting it into the victim's drink. Now, there is a new way to find out whether there is Benzodiazepine in a drink. AccuSign® DrinkChek is fast, easy, convenient, accurate, and one-step.

## Before You Begin

Please read this entire brochure before you begin testing. Store the test at 40-86°F (4-30°C) in the sealed pouch, and away from direct sunlight. Do not use after the expiration date stamped on the package.

Pad (sample application area)


Test Result Area

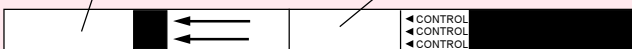

## Contents

- BZO test strip
- Disposable dropper
- Desiccant

## How to run test

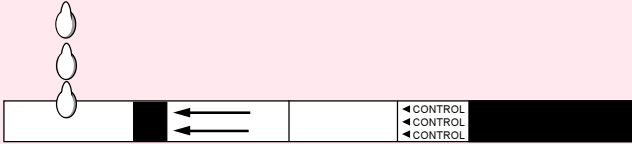

1. Remove the AccuSign® DrinkChek test strip from the sealed pouch, and place the strip on a flat surface.
2. Collect the sample of beverage to be tested using supplied disposable dropper.
3. Drop a small amount of collected beverage (2 drops of coffee, soda, etc.) on the sample pad of the strip, as in the diagram above. The test will not work properly if the liquid is added into the test result area or not correctly applied to the sample pad.
4. Read the result after three minutes, but within ten minutes.

## How to read results


A negative result (no Benzodiazepines) may appear in as early as 2 minute.

Negative: Both Control and Test lines appear in the Test result area. The intensity of the two lines may be different.

Positive: Only Control line appears in the Test result area.

Invalid: No Control line appears in the Test result area. The test did not work properly and the test should be repeated with a new strip. The strips are not reusable. If you have the same result(no control line) with the previous test, you may not need to test again since a certain component of the tested beverage may inhibit the test reaction.

## Examples of Possible Test Results:

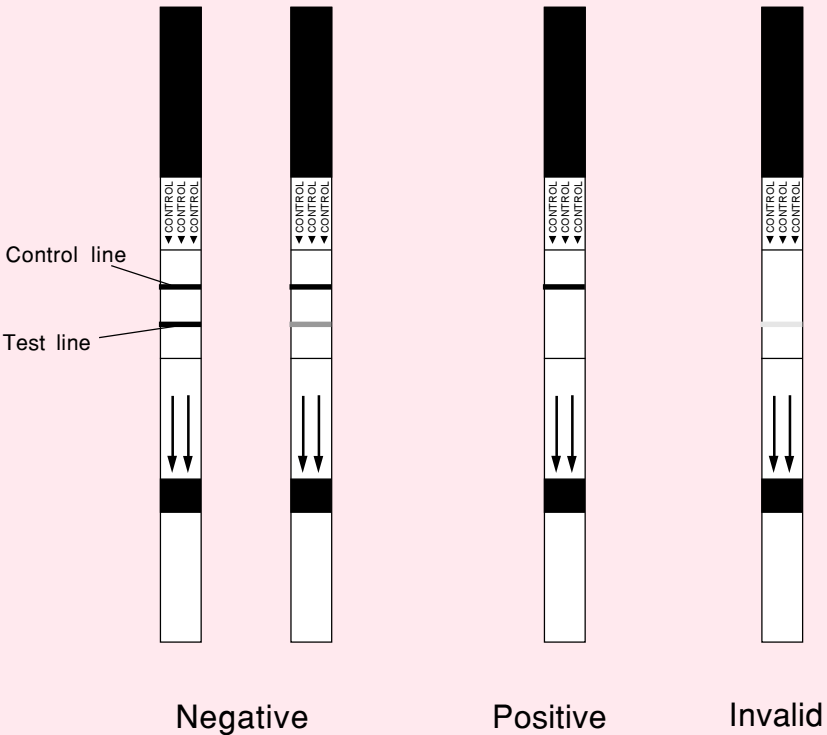

## How to interpret your test results

A negative result means no Benzodiazepines or less than 1ug per milliliter of Benzodiazepines was detected. The strength of line may vary depending on the tested drink.

A positive result means the drink contains Benzodiazepines above the cutoff level (1ug/ml of Flunitrozepam and/or Oxazepam in sample solution).

## Limitation

- This test is designed for use with beverages only.
- Test result read after 10 minutes may not be consistent with the original reading obtained within the 10 minute reading period. The test result must be read within 10 minutes of sample application.
- There is a possibility that factors such as technical or procedural errors, as well as other substances in the test beverage may interfere with the test reaction and cause erroneous result.
- The signal intensity (or color) may vary depending on the drink tested because of the color of the beverage itself.

## Assistance

If you have any question regarding the use of AccuSign® DrinkChek, please call our technical assistance number 1-800-726-2670 or 732-274-1000, Monday through Friday 9 A.M. to 5:30 P.M. Eastern Standard Time.

---

Manufactured by:



Princeton BioMeditech Corporation

Princeton, New Jersey 08543-7139 U.S.A.

Tel: (800) PBM-2670, (732) 274-1000 • Fax: (732) 274-1010

Internet E-mail: [support@pbmc.com](mailto:support@pbmc.com)

World Wide Web: <http://www.pbmc.com>