



Medicines and Healthcare products
Regulatory Agency

Evaluation Report MHRA 03078

Sixteen devices for the detection of drugs of abuse in urine

TABLE 1. Summary of analytical performance

	Amphetamines	Benzodiazepines	Cocaine metabolites	Methadone	Opiates	Cannabis
AccuSign® DOA 5						
Accuracy	☐	☆	☆		☆	☆
Imprecision	☐	☆	☆		☆	☆
AccuSign® MTD						
Accuracy				☆		
Imprecision				☆		
ACON® Multi-Drug 6 test Panel						
Accuracy	☆	☆	☆	☆	▲	☐
Imprecision	☐	☐	▲	☆	▲	☆
ACON® MOP morphine test device						
Accuracy					▲	
Imprecision					▲	
ADVANCED QUALITY™ ONE STEP Opiates (morphine) TEST						
Accuracy					☆	
Imprecision					▲	
Drug-Aware™ 6 Drug Urine Test-Type 2						
Accuracy	☆	☆	☆	☆	☐	☐
Imprecision	☐	☐	▲	▲	☆	☆
DrugSCREEN Multi Drug Test						
Accuracy	☐	☐	☆	▲	☆	☐
Imprecision	▲	☐	▲	▲	▲	☆
DrugSCREEN Platform						
Accuracy					☆	
Imprecision					☐	

Key: ☆ Good ☐ Satisfactory ▲ Poor

TABLE 1. Summary of analytical performance (continued)

	Amphetamines	Benzodiazepines	Cocaine metabolites	Methadone	Opiates	Cannabis
InstaCheck® Multi-drug Screen Panel						
Accuracy	☆	☆	☆	☆	☆	□
Imprecision	☆	▲	□	☆	☆	☆
Instant-View™ Multi-drug Screen Urine Test						
Accuracy	▲	□	▲	☆	☆	☆
Imprecision	□	□	▲	□	☆	☆
Instant-View™ Morphine (300) Urine Test						
Accuracy					☆	
Imprecision					▲	
SureScreen Multi-drug Screen Test						
Accuracy	▲	☆	☆	☆	▲	☆
Imprecision	□	▲	▲	☆	▲	☆
SureScreen MOP One Step opiate test Device						
Accuracy					☆	
Imprecision					▲	
SureStep™ Drug Screen Card III/300						
Accuracy	☆	☆	□	☆	▲	
Imprecision	▲	☆	▲	▲	▲	
SureStep™ Drug Screen test MOR 1						
Accuracy					▲	
Imprecision					▲	
Triage® 8						
Accuracy	☆	☆	☆	☆	☆	▲
Imprecision	□	▲	▲	☆	☆	□
Key: ☆ Good □ Satisfactory ▲ Poor						